

# SHUPACK

## SOFT A30

- Soft filling / shock absorbing material
- Stays soft even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time becomes hot during setting (in case of sensitive feet use SHUSIL)



**SHUPACK soft A30** is a soft fast setting hoofpad material offering more comfort for the horse. It can be used with or without pads for support as well as shock absorbing purposes. It can also give great benefits for foundered horses.

Comes in 250cc cartridges fitting in standard caulking guns. Fill up to ground level for support of foundered and laminitis cases. When used without pads, usage of meshes is recommended, but not obliged. Set time approx. 35 seconds. Temperature and humidity might have effect on the performance of this product.

### PRODUCT APPLICATION

#### Standard application

- With pads

#### Alternative application

- With mesh filled to ground level

### EAN CODE



### TECHNICAL PRODUCT DATA

<b>Product Number</b>	200130250
<b>Description</b>	Soft shock absorbing pad material with adhesion
<b>Cartridge</b>	250 cc
<b>Color</b>	Cristal Clear
<b>Set time</b>	35 seconds

#### Glue-U Adhesives BV

Droogdokkeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

#### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

EN  
V.003.EU

# SHUPACK

## MEDIUM A50

- Medium hard shock absorbing material
- Stays stable during shoeing period, even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time
- Becomes hot during setting (in case of sensitive feet use SHUSIL)



**SHUPACK A50** is a shock absorbing pad material with a setting time of 35 seconds often used for sport horses. It can be used with pad or mesh. For the best result it is advisable to use a mesh.

Comes in 250cc cartridges fitting in standard caulking guns. Fill up to ground level for support of foundered and laminitis cases. Usage of meshes is recommended, but not obliged. Set time approx. 35 seconds. Temperature and humidity might have effect on the performance of this product.

### PRODUCT APPLICATION

#### Standard application

- With pads

#### Alternative application

- With mesh

### EAN CODE



### TECHNICAL PRODUCT DATA

<b>Product Number</b>	200150250
<b>Description</b>	Medium hard & supportive pad material with adhesion
<b>Cartridge</b>	250 cc
<b>Color</b>	Cristal Clear
<b>Set time</b>	35 seconds

#### Glue-U Adhesives BV

Droogdokkenland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

#### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

**EN**  
**V.003.EU**

# SHUPACK

## CS A50

- Medium hard shock absorbing material
- Stays stable during shoeing period, even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time
- Protects frog against thrush



**SHUPACK CS** is a shock absorbing pad material with a setting time of 35 seconds often used for sport horses. It can be used with pad or mesh. For the best result it is advisable to use a mesh.

Comes in 250cc cartridges fitting in standard caulking guns. Fill up to ground level for support of foundered and laminitis cases. Usage of meshes is recommended, but not obliged. Set time approx. 35 seconds. Temperature and humidity might have effect on the performance of this product.

### PRODUCT APPLICATION

#### Standard application

- With pads

#### Alternative application

- With mesh

### EAN CODE



8 720929 179985

### TECHNICAL PRODUCT DATA

<b>Product Number</b>	200152250
<b>Description</b>	Medium hard & supportive pad material with adhesion and extra protection against thrush
<b>Cartridge</b>	250 cc
<b>Color</b>	Green
<b>Set time</b>	35 seconds

#### Glue-U Adhesives BV

Droogdokkeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

#### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

**EN**  
**V.003.EU**

# SHUGUARD

## A60

- Hard supporting pad material
- Stays stable during shoeing interval, even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time
- Becomes hot during setting



**SHUGUARD** is a supporting pad material which can be used on horses with shoes or barefoot. Thanks to the support on the sole it relieves the pressure on the wall and improves heel growth.

Comes in 250cc cartridges fitting in standard caulking guns. Set time approx. 35 seconds. Temperature and humidity might have effect on the performance of this product.

### PRODUCT APPLICATION

#### Standard application

- No pads needed, possible with mesh

### EAN CODE



### TECHNICAL PRODUCT DATA

<b>Product Number</b>	200161250
<b>Description</b>	Supportive pad material with adhesion
<b>Cartridge</b>	250 cc
<b>Color</b>	Black
<b>Set time</b>	35 seconds

#### Glue-U Adhesives BV

Droogdokeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

#### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

**EN**  
**V.003.EU**

# SHUSIL SOFT A10

- Very soft filling material
- Stays soft even in cold weather conditions
- No adhesion to the sole 4 minutes setting time
- If heated upto body temperature, setting time becomes 2 minutes
- Keeps low temperature during setting
- Acid free



**SHUSIL A10** is a silicone product to be used as hoofpad material. Can be used as filling or light shock absorbtion.

250cc cartridges fitting in standard caulking guns. Offers protection and light shock absorbtion for the hoof. Always use SHUSIL in combination with Shupads! SHUSIL does not contain acids or chemicals and are therefore very suitable in combination with wound treatment. Setting time 4 minutes, depending on weather conditions. Heating up SHUSIL before use speeds up setting time significantly. Temperature and humidity might have effect on the performance of this product.

## PRODUCT APPLICATION

### Standard application

- Always with pads

## EAN CODE



8 720929 170173

## TECHNICAL PRODUCT DATA

<b>Product Number</b>	210105250
<b>Description</b>	Extremely soft silicone pad material
<b>Cartridge</b>	250 cc
<b>Color</b>	Blue
<b>Set time</b>	4 minutes (heated 2 minutes)

### Glue-U Adhesives BV

Droogdokkenland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

**EN**  
**V.003.EU**

# SHUSIL

## MEDIUM SOFT A20

- Soft filling / light shock absorbing material
- No adhesion
- Stays soft even in cold weather conditions
- 4 minutes setting time (2 minutes when heated)
- Keeps low temperature during setting
- Acid free



**SHUSIL Medium Soft A20** is a transparent silicone product to be used as hoofpad material. It can be used as filling or shock absorption.

250cc cartridges fitting in standard caulking guns. Offers protection, shock absorption and support for the hoof. Always use SHUSIL in combination with Shupads! SHUSIL does not contain acids or chemicals and are therefore very suitable in combination with wound treatment. Setting time 4 minutes, depending on weather conditions. Heating up SHUSIL before use, speeds up setting time significantly. Temperature and humidity might have effect on the performance of this product.

### PRODUCT APPLICATION

**Standard application**

- Always with pads

### EAN CODE



### TECHNICAL PRODUCT DATA

<b>Product Number</b>	210103250
<b>Description</b>	Medium soft silicone pad material
<b>Cartridge</b>	250 cc
<b>Color</b>	Transparent
<b>Set time</b>	4 minutes (heated 2 minutes)

#### Glue-U Adhesives BV

Droogdokkeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

#### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

**EN**  
**V.003.EU**

# SHUSIL MEDIUM A30

- Medium shock absorption material
- No adhesion
- Stays soft even in cold weather conditions
- 4 minutes setting time (2 minutes when heated)
- Keeps low temperature during setting
- Acid free



**SHUSIL Medium A30** is a silicone hoofpad material, mostly used for shock absorption. It offers more comfort to sport horses and also reduces stress on the feet for foundered horses.

250cc cartridges fitting in standard caulking guns. Offers protection, shock absorption and support for the hoof. Always use SHUSIL in combination with Shupads! SHUSIL does not contain acids or chemicals and are therefore very suitable in combination with wound treatment. Setting time 4 minutes, depending on weather conditions. Heating up SHUSIL before use, speeds up setting time significantly. Temperature and humidity might have effect on the performance of this product.

## PRODUCT APPLICATION

### Standard application

- Always with pads

## EAN CODE



## TECHNICAL PRODUCT DATA

<b>Product Number</b>	210109250
<b>Description</b>	Medium silicone shock absorbing pad material
<b>Cartridge</b>	250 cc
<b>Color</b>	Purple
<b>Set time</b>	4 minutes (heated 2 minutes)

### Glue-U Adhesives BV

Droogdokeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: [sales@glue-u.com](mailto:sales@glue-u.com)  
Website: [www.glue-u.com](http://www.glue-u.com)

### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: [sales-us@glue-u.com](mailto:sales-us@glue-u.com)  
Website: [www.glue-u.com](http://www.glue-u.com)

**EN**  
**V.003.EU**

# SHUDIM AIR

## ULTRA LIGHT A20

- Ultra light pad material
- Non toxic
- Setting time 4-5 min (heated 2-3 min)



**SHUDIM Air** is an ultra light impression material to be mixed by hand. It's mostly used for filling, light shock absorption or just to give more comfort to the horse. Since the product is very soft it is only possible to fill complete hoofs.

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand. The ultra light SHUDIM Air is available in 2100 cc in durometer A20 and weighs 35% less than any standard impression material in the industry. SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals and is very suitable in combination with wound treatment. Temperature and humidity might have effect on the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.

### PRODUCT APPLICATION

**Standard application**

- Always with pads

### EAN CODE



### TECHNICAL PRODUCT DATA

<b>Product Number</b>	250114140
<b>Description</b>	Ultra light putty pad material mixed by hand
<b>Jar / Bucket</b>	2100 cc + 3x23 cc / 1,4 kg + 3x23 cc / 3.08 lb + 3x23 cc
<b>Color</b>	Orange
<b>Set time</b>	4-5 minutes (heated 2-3 minutes)

### Glue-U Adhesives BV

Droogdokkeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

**EN**  
**V.003.EU**



# SHUDIM

## SUPER SOFT A15

- Very soft impression material
- Non toxic
- Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the hoof



**SHUDIM Super Soft A15** is an extra soft impression material to be mixed by hand. It's mostly used for filling, light shock absorption or just to give more comfort to the horse. Since the product is very soft it is only possible to fill complete hoofs. In case of laminitis or other cases where there is an extremely painful area of the sole pad that needs to be isolated, the impression material can be trimmed away once it is cured after applying the shoe and pad.

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand. SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals and is very suitable in combination with wound treatment. Temperature and humidity might have effect on the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.

### PRODUCT APPLICATION

#### Standard application

- Always with pads

#### Alternative application

- Medical plate

### EAN CODE

660 gr



8 720929 170272

2500 gr



8 720929 170289

### TECHNICAL PRODUCT DATA

<b>Product Number</b>	250105660 (660 gr / 1.5 lb)   250105025 (2500 gr / 5.5 lb)
<b>Description</b>	Extra soft putty pad material mixed by hand
<b>Jar / Bucket</b>	2x660 gr / 2x1.5 lb   2x2500 gr / 2x5.5 lb
<b>Color</b>	Blue
<b>Set time</b>	4-5 minutes (heated 2-3 minutes)

### Glue-U Adhesives BV

Droogdokkeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

EN  
V.003.EU

# SHUDIM SOFT A25

- Soft impression material
- Non toxic Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the hoof



**SHUDIM Soft A25** is a soft impression material and it is the most used version of this product range. It gives a perfect shock absorption to the complete sole or locally as frog support in combination with for example a heartbar shoe.

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand. SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals and is very suitable in combination with wound treatment. Temperature and humidity might have effect on the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.

## PRODUCT APPLICATION

### Standard application

- Always with pads

### Alternative application

- Locally on bar shoes (Hearthbar, Straightbar)

## EAN CODE

660 gr



2500 gr



## TECHNICAL PRODUCT DATA

<b>Product Number</b>	250106660 (660 gr / 1.5 lb)   250106025 (2500 gr / 5.5 lb)
<b>Description</b>	Soft putty pad material mixed by hand
<b>Jar / Bucket</b>	2x660 gr / 2x1.5 lb   2x2500 gr / 2x5.5 lb
<b>Color</b>	Pink
<b>Set time</b>	4-5 minutes (heated 2-3 minutes)

## Glue-U Adhesives BV

Droogdokkeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

## Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

EN  
V.003.EU

# SHUDIM MEDIUM A35

- Medium impression material
- Non toxic Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the hoof



**SHUDIM Medium A35** is your best partner to support the hoof completely or just locally at the frog. Thanks to the rigid end product it's very easy to use with all kinds of barshoes or pads.

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand. SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals and is very suitable in combination with wound treatment. Temperature and humidity might have effect on the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.

## PRODUCT APPLICATION

### Standard application

- Always with pads

### Alternative application

- Locally on bar shoes (Hearthbar, Straightbar)

## EAN CODE

660 gr



2500 gr



## TECHNICAL PRODUCT DATA

<b>Product Number</b>	250107660 (660 gr / 1.5 lb)   250107025 (2500 gr / 5.5 lb)
<b>Description</b>	Medium putty pad material mixed by hand
<b>Jar / Bucket</b>	2x660 gr / 2x1.5 lb   2x2500 gr / 2x5.5 lb
<b>Color</b>	Green
<b>Set time</b>	4-5 minutes (heated 2-3 minutes)

## Glue-U Adhesives BV

Droogdokkeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

## Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

EN

V.003.EU

# SHUFIX WHITE

- Super fast setting time (30 - 60 seconds)
- Easy to rasp
- Hard but flexible material
- High temperature while setting (keep +/- 1 cm / 0.4") from coronary band)



**SHUFIX White** is an ultra fast urethane glue which is very suitable for making extensions on foals' hoofs and custom plastic shoes. Also suitable for retouching or rebuilding heels, making medial/lateral corrections and for smaller hoof repairs. In severe winter circumstances SHUFIX can also be used to glue most kind of glue-on shoes. Since it's a urethane product, the hoof should be dry and clean before applying SHUFIX.

250cc cartridges fitting in standard caulking guns. Curing time between 30 - 60 seconds, workable after about 5 minutes. Temperature and humidity can influence the performance of this product.

## PRODUCT APPLICATION

### Standard application

- Hoof extensions

### Alternative application

- Glue-on shoes
- Crack repairs
- Temporary protection
- Heel lifts and medial/lateral corrections

## EAN CODE



## TECHNICAL PRODUCT DATA

<b>Product Number</b>	150101250
<b>Description</b>	Instant crack repair super fast setting / foal extensions
<b>Cartridge</b>	250 cc
<b>Color</b>	White
<b>Set time</b>	30-60 seconds

### Glue-U Adhesives BV

Droogdokkeniland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

EN  
V.003.EU

# SHUBOND BLACK

- 150cc cartridges usable in standard standard caulking guns
- Very high adhesion
- Easy to rasp
- Hard but flexible material
- Setting time can be speed up when heated to 30 degrees Celsius



**SHUBOND Black** acrylic glue is the most versatile glue used in the horse industry. It can be used to glue-on every kind of shoe or do repairs from the smallest crack to the most extreme hoof wall problems.

Available in 150cc and 420cc. Fits in a standard caulking gun using an adapter or just use the Glue-U dispenser for 150 cc cartridges. 420cc only usable with special gun. Bonds steel, aluminium and any kind of glue-on shoe. Suitable for repairing large cracks and creating extensions.

Initial set time: 1,5-2 minutes (open time). Weight-bearing in 4-6 minutes (even in cold conditions). Temperature and humidity might have effect on the performance of this product. Heating up SHUBOND before use, speeds up setting time significantly.

## PRODUCT APPLICATION

### Standard application

- Glue-on shoes

### Alternative application

- Crack repairs
- Hoof repairs

## EAN CODE



## TECHNICAL PRODUCT DATA

<b>Product Number</b>	100102150 (150 ml) / 100102420 (420 ml)
<b>Description</b>	Glue-on shoe / Hoof repair
<b>Cartridge</b>	150 cc   420 cc
<b>Color</b>	Black
<b>Set time</b>	Initial set time: 1,5-2 minutes / Weight bearing: 4-6 minutes / Heated: 3 - 4 minutes

### Glue-U Adhesives BV

Droogdokkeniland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

**EN**  
**V.003.EU**

# SHUBOND

## BEIGE FAST SET

- Fast setting time / Weight bearing 3-4 minutes
- Easy to rasp
- Hard but flexible material
- Setting time can be speed up when heated to 30 degrees Celsius.



**SHUBOND Beige** acrylic glue is the most versatile glue used in the horse industry. It can be used to glue-on every kind of shoe or do repairs from the smallest crack to the most extreme hoof wall problems.

Available in 150cc and 420cc. Fits in a standard caulking gun using an adapter or just use the Glue-U dispenser for 150 cc cartridges. 420cc only usable with special gun. Bonds steel, aluminium and any kind of glue-on shoe. Suitable for repairing large cracks and creating extensions.

Initial set time: 1,5-2 minutes (open time). Weight-bearing in 3-4 minutes (even in cold conditions). Temperature and humidity might have effect on the performance of this product. Heating up SHUBOND before use speeds up setting time significantly.

### PRODUCT APPLICATION

#### Standard application

- Glue-on shoes

#### Alternative application

- Crack repairs
- Hoof repairs

### EAN CODE



### TECHNICAL PRODUCT DATA

<b>Product Number</b>	100114150 (150 ml) / 100114420 (420 ml)
<b>Description</b>	Glue-on shoe / Hoof repair
<b>Cartridge</b>	150 ml / 150 cc   420 ml / 420 cc
<b>Color</b>	Beige
<b>Set time</b>	Initial set time: 1,5-2 minutes / Weight bearing: 3-4 minutes / Heated: 2-3 minutes

#### Glue-U Adhesives BV

Droogdokkeniland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

#### Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

EN  
V.003.EU

# PRE FIX ACTIVATOR AND CA GLUE

- 5 seconds fast setting superglue
- Bonds once 2 components are in contact



**PRE FIX** is a system to bond a GLUSHU to the foot just to keep it in place. It is a 2 component superglue which will bond the GLUSHU to the hoof in seconds. This allows you to do the actual glueing of the shoe with the foot on the ground. PRE FIX is very useful if you have a very nervous horse or a horse that is unable to stand on three legs (laminitis or injury). PRE FIX is no structural glue. It is temporarily only to keep the GLUSHU in place for the time you need until the actual glue is cured. One component is the activator spray, the other component is a transparent gel in a squeeze bottle.

PRE FIX is used in 2 steps. Once you fitted and shaped the GLUSHU and you are ready to glue, apply the PRE FIX CA GLUE on the GluShu and spray the activator on the hoof. After pre fixing the shoe you can further finish the glueing on the ground with SHUBOND for permanent fixation.

## PRODUCT APPLICATION

### Standard application

- Only for pre-application of GLUSHU shoes

## EAN CODE

Activator



8 720929 170562

Ca Glue



8 720929 170579

## TECHNICAL PRODUCT DATA

### Activator

<b>Product Number</b>	490100200
<b>Description</b>	Pre glue GLUSHU in combination with PRE FIX CA GLUE
<b>Cartridge</b>	200 cc
<b>Color</b>	Transparent
<b>Set time</b>	5 sec

### CA Glue

<b>Product Number</b>	490101150
<b>Description</b>	Pre glue GLUSHU in combination with PRE FIX ACTIVATOR

## Glue-U Adhesives BV

Droogdokkeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: sales@glue-u.com  
Website: www.glue-u.com

## Glue-U USA Inc

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: sales-us@glue-u.com  
Website: www.glue-u.com

**EN**  
**V.003.EU**

<b>Cartridge</b>	150 cc
<b>Color</b>	Transparent
<b>Set time</b>	5 sec

**Glue-U Adhesives BV**

Droogdokeneiland 8  
5026 SR Tilburg  
The Netherlands

Tel. +31 (0)13 545 31 18  
E-mail: [sales@glue-u.com](mailto:sales@glue-u.com)  
Website: [www.glue-u.com](http://www.glue-u.com)

**Glue-U USA Inc**

215 Church Street  
Georgetown  
SC 29440 USA

Tel. +1 (313) 509 - 7766  
E-mail: [sales-us@glue-u.com](mailto:sales-us@glue-u.com)  
Website: [www.glue-u.com](http://www.glue-u.com)

**EN**  
**V.003.EU**